

GUIDE TO INTELEVIEWER & INTELECONNECT EV



ACCOUNT ACTIVATION

IntelViewer:

(Diagnostic image viewer - icon on your desktop)

Black and red log-in screen. Requires installation.




Installation

1. If you already have IntelViewer installed with our Queensland X-Ray Bookmark, click on the IntelViewer Icon. 
2. Enter your username and password.
3. If this is the first time you have accessed IntelViewer, you will be directed to the IntelBrowser where you will need to accept the T&C's. See following link: <https://qxrpacks.com.au/IntelBrowser>
4. Once you have read and agreed to the T&C's, log out of this page and return to the IntelViewer log in page.
5. Enter username and password and enter.
6. You will now be logged into IntelViewer.
7. If you do not have IntelViewer installed select the link below then select the download you require: <https://www.qldxray.com.au/referrers/image-report-delivery/inteleviewer/> ie: 32bit, 64bit or MAC.

ACCOUNT ACTIVATION

✓ Once the download is complete

1. Double click on the IntelViewer Icon. You will be prompted to add the Server URL Address (bookmark).
2. Enter <https://qxrpacs.com.au>
3. Select **'Protect Bookmark'**.
4. Select **'Use Custom Server Name'** and name it: **Queensland X-Ray**.
5. Select **Ok** and **Close**.

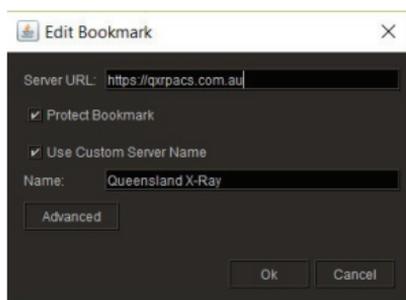


+ Adding Queensland X-Ray Bookmark if IntelViewer has already been installed by another Radiology company:

1. Double click on the IntelViewer Icon.
2. Select **'Edit Bookmarks'**.



3. Select **'Add'**.
4. Enter <https://qxrpacs.com.au>
5. Select **'Protect Bookmark'**.
6. Select **'Use Custom Server Name'** and name it: **Queensland X-Ray**.
7. Select **Ok** and **Close**.

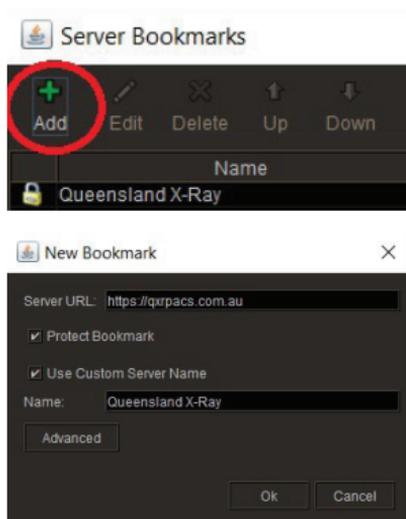


ACCOUNT ACTIVATION

IntelConnect EV:

(Diagnostic image viewer via web browser)

Blue and white log-in screen. No installation required. Browser based. Available on all smart phones, laptops, tablets and computers.

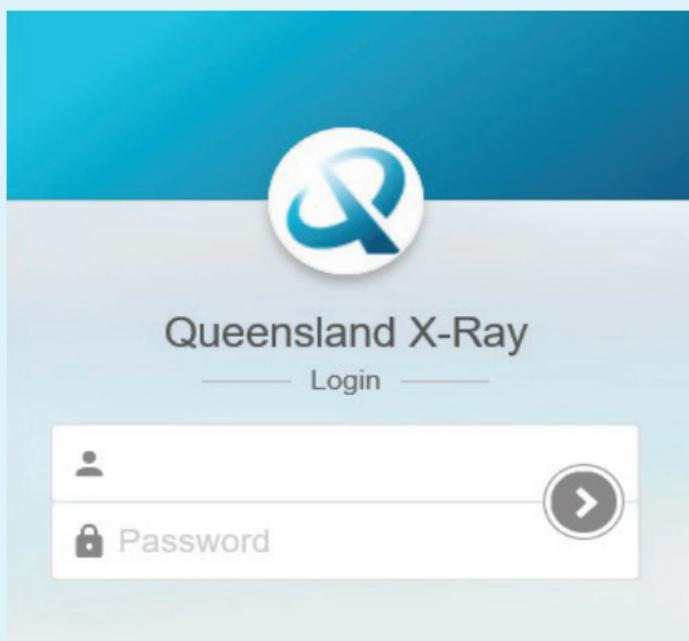

Set Up

1. Go to: <https://qxrpacs.com.au/portal/app>
2. Enter your username and password. If this is the first time you have accessed IntelConnect EV, you will be directed to the IntelBrowser where you will need to accept the T&C's. <https://qxrpacs.com.au/IntelBrowser>
3. Once you have read and agreed to the T&C's, log out of this page and return to the IntelConnect EV log in page.
4. Enter username and password and enter.
5. You will now have access to IntelConnect.

MOST COMMONLY USED TOOLS

Please note: In both programs if you hover your mouse cursor over the icon it will tell you what their function is.

.....

InteleViewer:

Patient Search




If you have referred the patient:

1. Select the Search icon as above.
2. In the Patient Name Field. Enter Last name, First name and enter. (Eg. Smith, John).

Note: If you are not the referring doctor, you will need to access the Break Glass Access (see instructions below).

.....

Break Glass Access

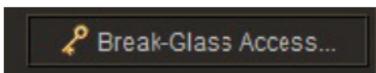

If you are not the referring doctor:

1. Select '**Break Glass Access**' (right hand side of screen).
 2. Read and accept the T&C's.
 3. To search using patient ID, enter QXR ID and select enter.
 4. To search using patient name, enter Last name, First name. (Eg. Smith, John).
 5. Enter D.O.B. YYYYMMDD. (eg: 19620312)
No dots or slashes.
 6. Select '**Access Patient Studies**'. All prior studies will be pushed through for viewing from 2011.
-

View Report



1. After selecting a patient study, select the View Reports icon as above and the reports will appear.
-

Multiple image viewing

1. To view more than one image at a time. Using your keyboard numbers, select 1,2,3,4 etc...
2. Drag and drop relevant studies down to the viewing window.

MOST COMMONLY USED TOOLS

Other useful tools



Magnify

Adjust brightness and contrast

Zoom image

Pan image

Move image around in the window.

Anonymising studies


IntelConnect EV:

Patient Search

1. Enter Last name, First name.
(Most recent studies will automatically appear).
2. Double click on relevant study.
(Report and images will appear side by side).

Break Glass Access IntelConnect EV

Please note: To Break Glass in IntelConnect EV you will need to click on the padlock before adding the name.

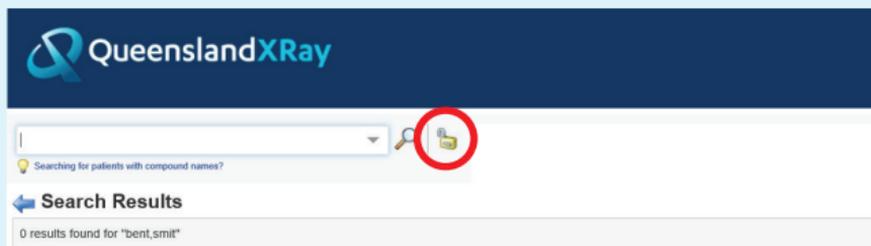

Other useful tools



Adjust brightness and contrast



Zoom image



Pan image

Move image around in the window.



Multiple image viewing



Anonymising studies